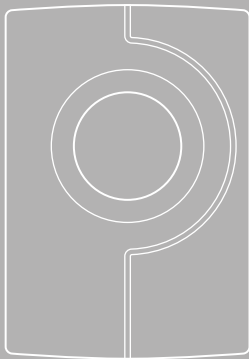
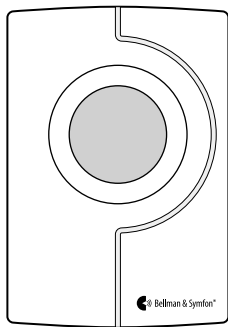
BE1240-433/BE1420  
Visit Pushbutton Transmitter

## Visit Pushbutton Transmitter

- |    |                         |
|----|-------------------------|
| 4  | User manual             |
| 14 | Manual del usuario      |
| 24 | Manuel d'utilisation    |
| 34 | 用戶手冊                    |
| 44 | 用戶手冊                    |
| 54 | ユーザーマニュアル               |
| 64 | Appendix (English only) |

EN

ES

FR

CN<sub>SI</sub>

CN<sub>TM</sub>

JP

App

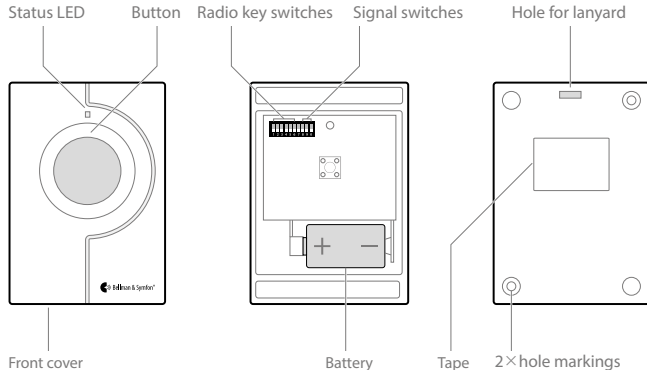
## Read this first

This booklet guides you on how to use and maintain your new Pushbutton Transmitter. Ensure you read this booklet carefully, including the **Warnings** section. If you have additional questions, contact your point of purchase for further assistance.

**Intended user group** The intended user group are people of all ages.

**Principle of operation** The Visit alerting system consists of a set of wirelessly connected transmitters and receivers that are located across the home. When a transmitter detects an activity, it signals the Visit receiver that alerts the user with sound, flashes or vibrations. In order to use the system, you need at least one transmitter and one receiver.

## Overview


EN

## Using it as a doorbell
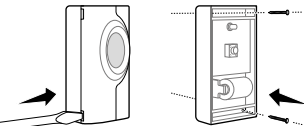
### Mounting with screws

- 1 Remove the transmitter front cover and make two holes on the markings.
- 2 Fix the unit to the wall using the supplied screws and put the cover back on.

**Mounting with tape.** You can instead choose to use the supplied adhesive tape. Attach it to the back of the transmitter and mount it in a weather protected area.


## Using it as a caller button

The transmitter can be worn around your neck and be used as a wireless caller button.

### Here is how you use it:

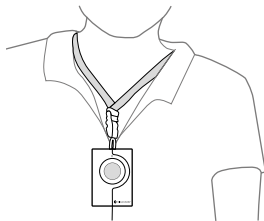Attach the lanyard to the hole on the top off your device and hang it around your neck.



## Testing the signal

### Using the test button

- 1 Press the front button on the transmitter. The LED lights up in green to show that a radio signal is being transmitted.
- 2 The green Visit LED on the receiver lights up to show that the signal was received. In addition, it starts to sound, flash or vibrate depending on the receiver.

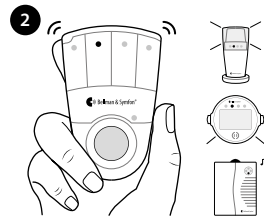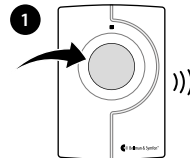**Note:** If you have changed the signal pattern, it will react in accordance with the signal tables that you can find in the Appendix (English only).

## General warnings

This section contains important information about safety, handling and operating conditions. Keep this booklet for future use. If you are just installing the device, this booklet should be given to the householder.

### Hazard warnings

- Failure to follow these safety instructions could result in fire, electric shock, or other injury or damage to the device or other property.
- Make sure to keep this device out of reach of children under 3 years.
- Note that alarms and notifications can be missed if batteries run out.
- Only use battery types that are specified in this leaflet.
- This device is for indoor use and outdoor use in a protected location. It will not withstand water or rain.
- Protect the device from shocks during storage and transport.
- Do not use or store this device near any heat sources such as naked flames, radiators, ovens or other devices that produce heat.
- This equipment radiates radio frequency energy. If not installed and used in accordance with the user guide, may cause harmful interference to radio communication.
- Do not dismantle the device; there is a risk of electric shock. Tampering with or

dismantling the device will void warranty.

- Do not make any changes or modifications to this device. Use only original Bellman & Symfon accessories to avoid any electrical shock.
- Batteries are toxic. Do not swallow them! Keep out of reach of children and pets. If they are swallowed, consult your physician immediately!

### Information on product safety

- Failure to follow these instructions could result in damage to the device and void the warranty.
- Do not expose batteries to fire or to direct sunlight.
- Do not drop your device. Dropping onto a hard surface can damage it.
- Do not use the device in areas where electronic equipment is prohibited.
- The device may only be repaired by an authorized service center.
- If you encounter problems with the device, contact the point of purchase, your local Bellman & Symfon office or the manufacturer. Visit **bellman.com** for contact info.
- No company, enterprise or user is allowed to change the frequency, enhance the transmitting power or alter the original characteristics or performance on an approved low power radio frequency device.
- This device should not be used on aircrafts unless specifically permitted by flight personnel.

**Cleaning**

Use a soft, lint-free cloth. Avoid getting moisture in openings. Do not use household cleaners, aerosol sprays, solvents, alcohol, ammonia, abrasives or compressed air.

**Service and support**

If the product appears to be damaged or doesn't function properly, follow the instructions in this booklet. If the product still doesn't function as intended, contact your local hearing care professional for information on service and warranty.

**Warranty conditions**

Bellman & Symfon guarantees this product (excluding the battery) for two (2) years from date of purchase against any defects that are due to faulty materials or workmanship. This guarantee only applies to normal conditions of use and service, and does not include damage resulting from accident, neglect, misuse, unauthorized dismantling, or contamination howsoever caused. This guarantee excludes incidental and consequential damage. Further the warranty does not cover Acts of God, such as fire, flood, hurricanes and tornadoes. This warranty gives you specific legal rights and you may also have other rights that vary from state to state. Some states or jurisdictions do not allow the limitation or exclusion of incidental or consequential damages, or limitations on how long an implied warranty last so the above limitation may not apply to you. This guarantee is in addition to your statutory rights as a consumer. The above warranty may not be altered except in writing signed by both parties hereto.

**Symbol explanation**

|                                                                                     |                                                                                                                                                                                                                                     |
|-------------------------------------------------------------------------------------|-------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
|  | Indicates the manufacturer's serial number so that a specific device can be identified.                                                                                                                                             |
|  | Indicates the device manufacturer.                                                                                                                                                                                                  |
|  | Indicates the need for the user to consult the instructions for use.                                                                                                                                                                |
|  | Indicates that caution is necessary when operating the device or control close to where the symbol is placed, or that the current situation needs operator awareness or operator action in order to avoid undesirable consequences. |
|  | Identifies a place where information may be found, especially in an emergency.                                                                                                                                                      |
|  | Indicates the temperature limits to which the device can be safely exposed.                                                                                                                                                         |
|  | Indicates the range of humidity to which the device can be safely exposed.                                                                                                                                                          |

# Troubleshooting

| If                                                           | Try this                                                                                                                                                                                                                                                                                                     |
|--------------------------------------------------------------|--------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| The LED lights up in yellow when I press the button          | <ul style="list-style-type: none"><li>▪ The battery is nearly depleted. Replace it with a new battery, see <b>Technical specifications</b> for battery type.</li></ul>                                                                                                                                       |
| The LED doesn't light up when I press the button             | <ul style="list-style-type: none"><li>▪ Check that the battery is positioned correctly.</li><li>▪ Replace it with a new battery, see <b>Technical specifications</b> for battery type.</li></ul>                                                                                                             |
| The LED lights up in green but the receiver is not activated | <ul style="list-style-type: none"><li>▪ Check the receiver batteries and connections.</li><li>▪ Move the receiver closer to the transmitter to make sure it's within radio range.</li><li>▪ Check that the units are set to the same radio key, see <b>Changing the radio key</b> in the Appendix.</li></ul> |
| The receiver is activated for no apparent reason             | <ul style="list-style-type: none"><li>▪ There is probably another Visit system installed nearby that triggers your system. Change the radio key on all units, see <b>Changing the radio key</b> in the Appendix.</li></ul>                                                                                   |

# Technical specifications

|                   |                                                                                                                                                                               |
|-------------------|-------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| Dimensions        | 66 × 48 × 23 mm, 2.6 × 1.9 × 0.9"                                                                                                                                             |
| Weight            | 50 g, 1.8 oz. including battery                                                                                                                                               |
| Power and battery | Battery: 1 × 6V alkaline PX28M/IEC-4LR44 or PX28L lithium<br>Power consumption: Active 35 mA, idle position 0.05 µA<br>Operating time: Alkaline ~ 2½ years, Lithium ~ 4 years |
| Activation        | Via the front button                                                                                                                                                          |
| Max output power  | -0,9 dBm                                                                                                                                                                      |
| Radio frequency   | 433.92 MHz                                                                                                                                                                    |
| Coverage          | 70 - 120m, 230-394 ft (433Mhz) / 30-80m (315Mhz), depending on the building's characteristics.                                                                                |
| Operation         | Temp: -10° to 40° C, Humidity: 15% – 90%, non-condensing                                                                                                                      |
| Configuration     | This transmitter can be used with all Visit receivers.                                                                                                                        |
| No. of radio keys | 64 as standard. Up to 256 with optional software.                                                                                                                             |
| In the box        | <ul style="list-style-type: none"><li>▪ BE1240-433 Visit Pushbutton Transmitter</li><li>▪ Adhesive tape, screws and plugs</li><li>▪ Lanyard with safety clasp</li></ul>       |

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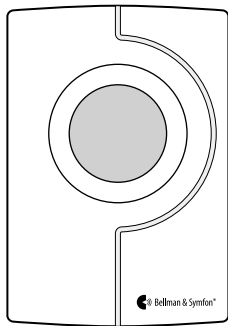
## Appendix (EN)

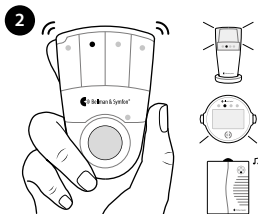
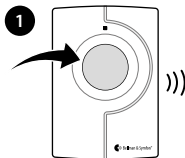
- 
- |    |                    |
|----|--------------------|
| 66 | Testing the signal |
|----|--------------------|
- 
- |    |                        |
|----|------------------------|
| 67 | Default signal pattern |
|----|------------------------|
- 
- |    |                             |
|----|-----------------------------|
| 67 | Changing the signal pattern |
|----|-----------------------------|
- 
- |    |                        |
|----|------------------------|
| 72 | Changing the radio key |
|----|------------------------|
-

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**Note:** If you have changed the signal pattern, it will react in accordance with the signal pattern tables on the following pages.

## Default signal pattern

When the Pushbutton Transmitter is activated, the following happens:

- 1 The LED on the transmitter lights up in green to show that it is signalling the receiver.
- 2 The Visit LED on the receiver lights up in green and it starts to sound, flash or vibrate with a certain pace, called signal pattern. The transmitter and the connected accessories determine the signal pattern. The default is as follows:

### Transmitter

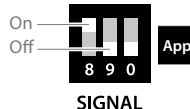
**LED**  
Green, top

### Visit receiver

| LED         | Sound               | Vibration | Flash |
|-------------|---------------------|-----------|-------|
| Green light | 1 × Door chime, low | Yes       | Yes   |

## Changing the signal pattern

Changing the signal pattern is easy. Open the transmitter front cover and move signal switches no. 8, 9 and 0 (315 MHz units) or 7, 8 and 9 (433 MHz units) according to the tables below:

## Transmitter Receiver signal pattern 315 MHz

| Switch                                                                            | LED                           | Sound              | Vibration | Flash |
|-----------------------------------------------------------------------------------|-------------------------------|--------------------|-----------|-------|
|  | Green light<br><b>Default</b> | 1×Door chime, low  | Slow ■□□□ | Yes   |
|  | 2×Green blinks                | 2×Door chime, low  | Slow ■□□□ | Yes   |
|  | 3×Green blinks                | 1×Door chime, high | Slow ■□□□ | Yes   |
|  | Green blinks                  | 2×Door chime, high | Slow ■□□□ | Yes   |

## Transmitter Receiver signal pattern 315 MHz

| Switch                                                                              | LED           | Sound               | Vibration   | Flash |
|-------------------------------------------------------------------------------------|---------------|---------------------|-------------|-------|
|  | Orange light  | Baby melody         | Fast ■■■■■■ | Yes   |
|  | Orange blinks | Baby melody         | Fast ■■■■■■ | Yes   |
|  | Yellow light  | 1×Ring signal, low  | Medium ■□■□ | Yes   |
|  | Yellow blinks | 2×Ring signal, high | Medium ■□■□ | Yes   |

App

## Transmitter Receiver signal pattern

433 MHz

| Switch                                                                            | LED          | Sound                | Vibration                                                                                  | Flash |
|-----------------------------------------------------------------------------------|--------------|----------------------|--------------------------------------------------------------------------------------------|-------|
|  | Red light    | 2× Ring signal, high | Medium  | Yes   |
|  | Orange light | 2× Ring signal, low  | Medium  | Yes   |
|  | Green light  | 1× Ring signal, high | Medium  | Yes   |
|  | Yellow light | 1× Ring signal, low  | Medium  | Yes   |

## Transmitter Receiver signal pattern

433 MHz

| Switch                                                                              | LED                    | Sound               | Vibration                                                                                | Flash |
|-------------------------------------------------------------------------------------|------------------------|---------------------|------------------------------------------------------------------------------------------|-------|
|  | Red light              | 2× Door chime, high | Long  | Yes   |
|  | Orange light           | 2× Door chime, low  | Long  | Yes   |
|  | Yellow light           | 1× Door chime, high | Long  | Yes   |
|  | Green light<br>Default | 1× Door chime, low  | Long  | Yes   |

App

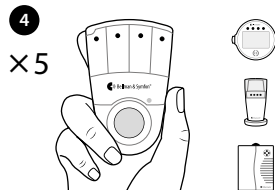
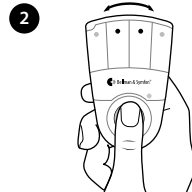
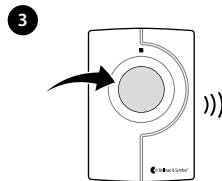
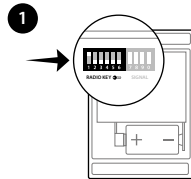
## Changing the radio key

If your Visit system is activated for no reason, there is probably a nearby system that triggers yours. In order to avoid radio interference, you need to change the radio key on all units. The radio key switches are located under the transmitter cover.

### Here is how you change the radio key:

- 1 Carefully remove the front cover on the push button transmitter. Move any radio key switch to the up (on) position to change the radio key.
- 2 Press and hold the test/function button on the receiver until the green and yellow Visit LEDs blink alternately. Release the button.
- 3 Press the button on the transmitter within 30 s to send the new radio key.
- 4 All Visit LEDs on the receiver blink alternately to show that the radio key has been changed. It then returns to normal mode. If you have a BE1033 Portable Receiver, the power LED blinks in amber to confirm.

**Please note:** All Visit units must be set to the same radio key.

## FCC compliance statement

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) this device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

**Warning:** Changes or modifications to this unit not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

**Note:** This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

## IC compliance statement

This device contains licence-exempt transmitter(s)/receiver(s) that comply with Innovation, Science and Economic Development Canada's licence-exempt RSS(s). Operation is subject to the following two conditions: (1) This device may not cause interference. (2) This device must accept any interference, including interference that may cause undesired operation of the device.

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**FCC ID:** WMSBETX2 **IC:** 6693A-BETX2

## Product disposal and recycling information

According to local laws and regulations your product should be disposed of separately from household waste. When this product reaches its end of life, take it to a collection point designated by local authorities. The recycling of your product will help conserve natural resources and ensure that it is recycled in a manner that protects human health and the environment.

## Battery disposal information

This product contains batteries that according to local laws and regulations must be disposed of separately from household waste. Take used batteries to a collection point designated by the authorities.

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All rights reserved.
**Bellman & Symfon North America Inc.**
**Bellman & Symfon Asia Ltd.**
Bellman & Symfon Group AB
TM and © 2025 Bellman & Symfon AB.