

User Guide
BE3330 Watch Receiver

Bellman.



Welcome!

Read this first

This booklet guides you on how to use and maintain your Watch Receiver from Bellman. Ensure you read this booklet carefully, including the Warnings section. If you have additional questions, contact your point of purchase for support.

Wearing your watch

Wear your watch firmly and comfortably around the wrist as shown in the figure. Wearing your watch with the right fit – not too tight, not too loose and with room for your skin to breathe – will keep you comfortable and allow the watch to generate distinct vibrations when something happens that needs your attention.



Contents

2	Get to know your watch
3	Charging your watch
4	Waking up the screen
5	Setting up your watch
9	The watch face
10	The menu
11	Do not disturb
12	Using the timer
13	Managing alarms
15	Managing notifications
17	Adding and removing devices
19	Settings menu
22	Signal for attention
25	Troubleshooting

Get to know your watch

Keeps tabs on your home

This Watch Receiver is part of the alerting system from Bellman, and it will help you notice important signals in your home. So, for instance, when someone rings your doorbell, you will get a notification on your watch that you have a visitor.







How it works

When e.g. your doorbell rings, the Visit Door Transmitter is activated and signals your watch.

Your watch alerts you with vibrations and shows you what's going on.


Tailor it to your needs


The system offers a variety of setups to help you keep a watchful eye on your home and loved ones.

-  Get emergency notifications from connected smoke and CO detectors.
-  Use doorbell sensors to notify you if someone rings your doorbell.
-  Use a baby monitor to know when your child needs your attention.
-  Get an alert on your watch when you have an incoming call or message.

Charging your watch

Charge the battery before you use your watch for the first time, or when you haven't used it for an extended period.

 The first time you charge your watch; it will start automatically, and you will be asked to select language. The battery indicator will not be visible initially.

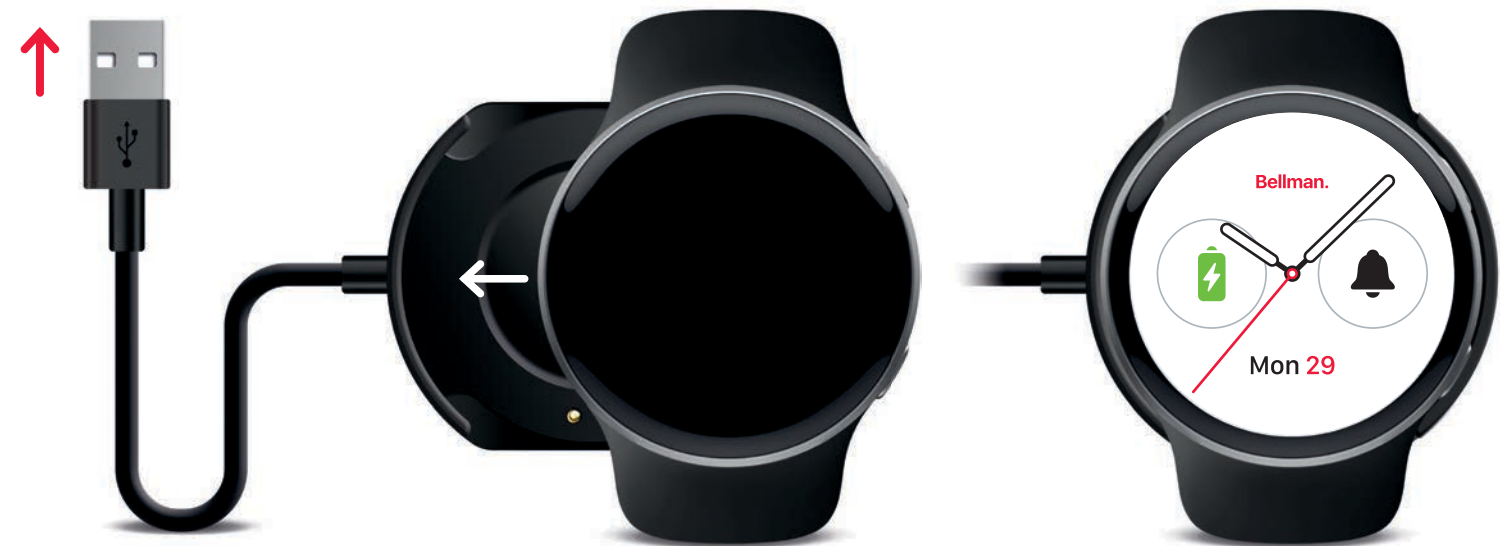

Connect to power

Connect the charging cable to an USB A power adapter or a USB A computer port.

Charge the watch

Attach your watch to the charging dock, making sure that the charging terminals ● ● align. It will start automatically.

On your watch dial

The battery indicator on the watch face turns green to show that it's being charged. It can take up to 2 h to fully charge it.

Waking up the screen

Using gestures


Normally you'll leave your watch on all the time and use your wrist to wake up the screen and put it to sleep.

- To wake up the screen, raise the wrist.
You can also press any of the side keys.
- To turn off the screen backlight, lower the wrist.
You can also press the Home key briefly.





Using the keys

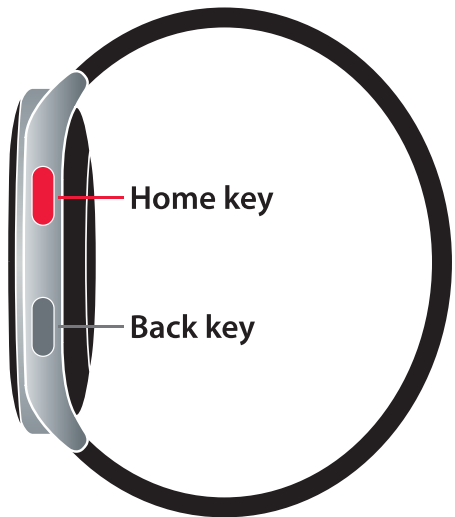

If you plan not to use your watch for a while, hold down the Home key to turn your watch off to save energy. If your watch has been turned off for an extensive period, it may revert to its default time. When you charge your watch, it will need up to 30s to synchronize with the system.



The Home key

-  Press to go back to the watch face.
-  Hold down to turn your watch on or off.

The Back key

-  Press to open the menu or go back one step.
-  Hold down for 4s to call for attention.

Setting up your watch

When you turn on the watch for the first time or reset it, follow the steps below to pair it to your system.


Before you begin, make sure the units are close to each other.

Pairing your watch



Start the bridge

Unpack the bridge and re-move the battery pull tab to start it. The top LED will start to pulsate slowly in blue.

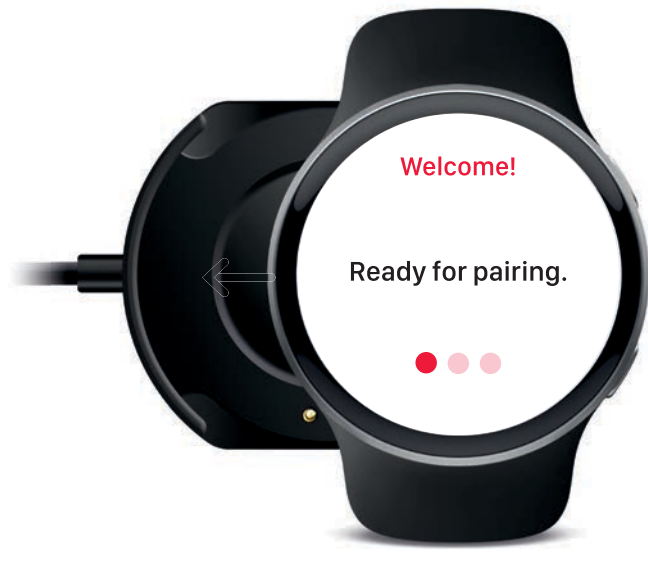


Charge your watch

Unpack your watch and place it in the charger for 15 min. Remove it from the cahrger, select your language and your watch is ready for pairing.



Pair your watch

Hold down the bridge **top** test button marked until the LEDs start to blink alternately. Your watch will let you know when it has been added to the system. Tap **OK**.

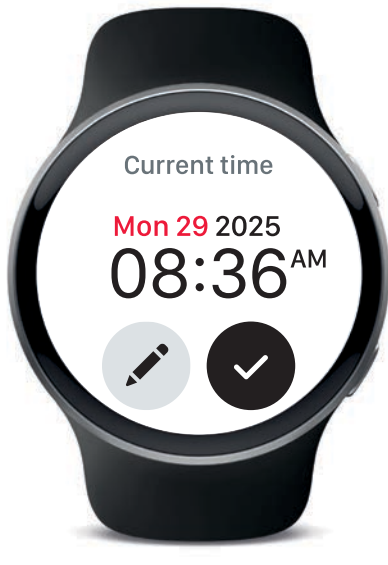

Showing the current time

If you are using the Assistant* app, the system time is shown. Tap **Edit** to fine tune the time manually or tap **Done** to complete the installation.

Setting the time manually

If you are not using the **Assistant app**, you will be asked to set the time and date manually.



Select time format

Select between 12-hour and 24-hour time format and tap **Next** to set the time.




Set the time

Select the hours or minutes and tap the red arrows to adjust the time. Tap **Next** to set the year.



Set the year

Tap the red arrows to set the year and tap **Next** to set the date.

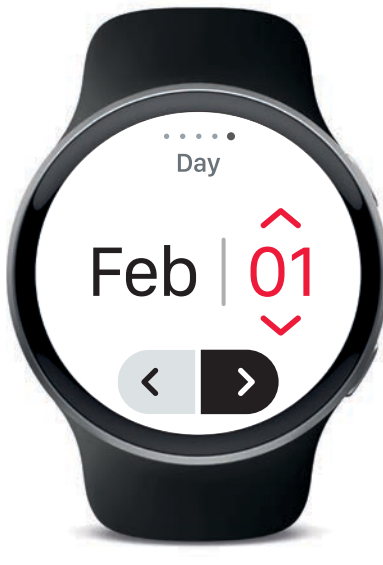

Set the date

Select the month or day and tap the red arrows to set the date. Tap **Next** to complete the installation.

* See page 8

Testing the connection


Make sure to test the connection after you have completed the installation, or when you haven't used your system for an extended period.

If the connection fails, go to the 'Troubleshooting' section for guidance.

To your watch and mobile phone



On your bridge

Press the **top** test button marked . The top LED lights up in blue to show that a signal is being transmitted.


On your watch and mobile

A message appears on your watch and mobile phone to confirm that the units are connected to your bridge.

To your Visit system



On your bridge

Press the **bottom** test button marked . The bottom LED lights up in green to show that a signal is being transmitted.

On your receiver

The yellow LED lights up and it starts to sound, flash or vibrate depending on your type of receiver.

The Assistant app

We recommend that you take the time to install the Assistant app from Bellman. It offers seamless communication between your Visit system and your mobile phone — making it easier than ever to keep tabs on your home.

Once you have installed the app, you will begin to receive Visit notifications on your mobile phone. In addition, your Visit receiver will alert you to incoming calls and messages.

Easy synchronization


Another benefit is that your Watch Receiver will synchronize the time with your mobile phone automatically, making it unnecessary to set the time, time zone and date manually.

The Assistant app from Bellman is free of charge and available for both iOS- and Android devices.

Install the Assistant app




Download the **Assistant** app from the App Store® or Google Play™ and follow the simple setup instructions.

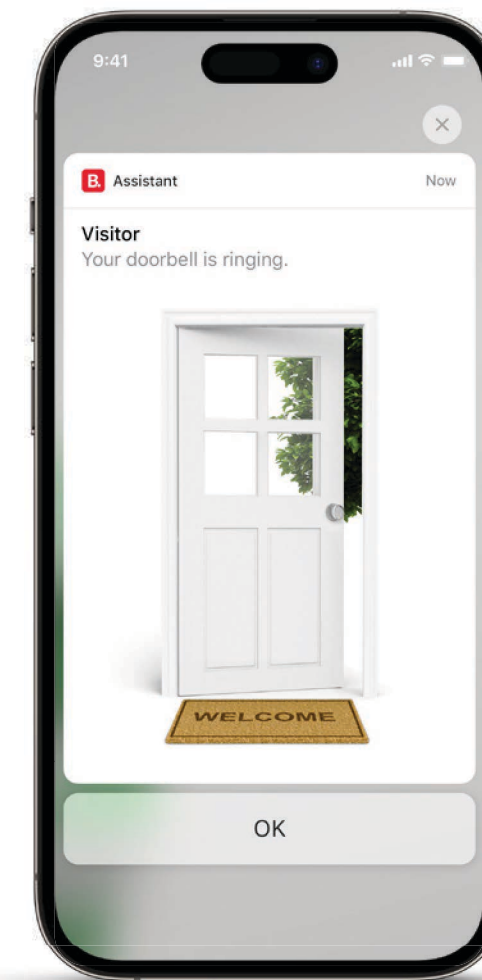

Clear notifications

Enjoy clear and distinct notifications on your mobile phone, tablet and even your smart watch.

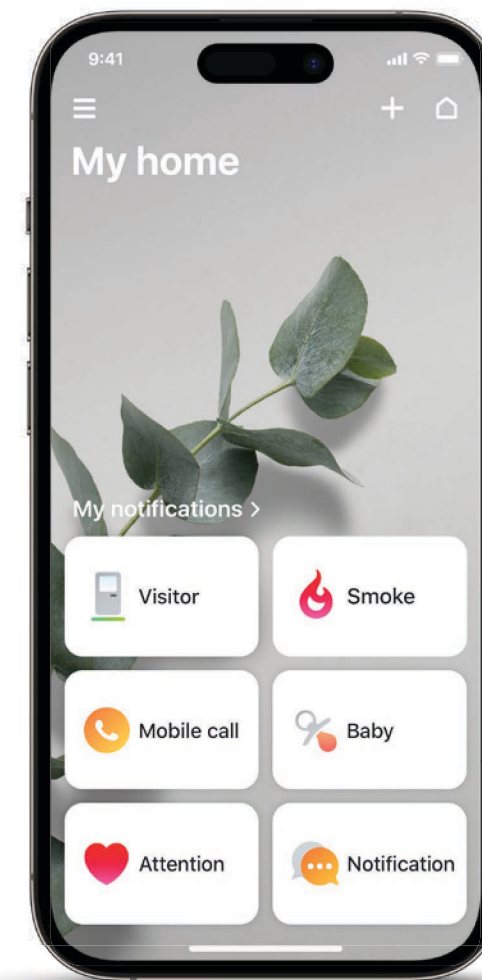






Easy to use

The home screen provides an overview of notifications, to and from your phone. Tap one to temporarily turn it off.

The watch face

Your watch receiver offers a selection of watch faces to choose from. No matter what design you choose, the watch will feature three complications: Date, Battery status and Notification history. But if the watch is out-of-range or if the notifications are turned off, red symbols will temporarily take their place to notify you.

Overview






	Date Displays the day and date.
	Battery status Indicates the current battery level.
	Notification history Shows the number of missed notifications.
	Out-of-range Warns when your watch is out-of-range.
	Do Not Disturb Means that the notifications are turned off.



The menu

In the menu you have access to a multitude of useful features like for instance, a timer and an alarm clock. Here, you can also fine tune your watch to match your needs. To open the menu, press the **Back key** on the right side of the watch.

Overview

	Page
	Do Not Disturb Find out how to turn off a notification temporarily. 11
	Timer Get to know the powerful timer function. 12
	Alarm Learn how you turn your watch into an alarm clock. 13
	Night Mode Discover how to reduce eye strain during the night. 15
	Settings Tailor your watch to fit your needs. 19



Do Not Disturb

This function will temporarily silence the notifications on your watch. Just pick a preset like e.g. 10, 20, or 30 minutes to activate it. Please note that fire and CO notifications cannot be silenced for security reasons.

To open the Do Not Disturb panel, press the **Back key** and tap **Do Not Disturb**  in the menu.




In the Do Not Disturb panel

Tap to pick a preset duration, like for instance 10, 20 or 30 minutes.




On your watch dial

The Do Not Disturb symbol  shows that it is activated.
Tap the symbol to turn the notifications on again.



When the time is up

The Notifications symbol  re-appears to show that the notifications are turned back on.


Using the timer

The timer can help you keep track of time. You can create a custom timer or select a preset like e.g. 5, 10 or 15 minutes. If you pick a preset, the timer will start right away.

To open the Timer panel, press the **Back key** and tap **Timer**  in the menu.




In the Timer panel

Pick a preset timer, displayed in minutes, or tap **Add**  to create a custom timer.




Set the timer duration

Tap to select the hours or minutes and tap the arrows to adjust the time. Tap **Start**  to activate the timer.





On your watch dial


The **Timer**  shows the remaining time.
Tap the timer symbol to pause, resume or delete it.



When the time is up

- Tap  to restart the timer.
- Tap  to delete the timer and return to the watch face.


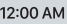
Managing the alarm

This section describes how to set, snooze, and turn an alarm On/Off.
To open the Alarm panel, press the **Back key** and tap **Alarm**  in the menu.

Setting an alarm




In the Alarm panel

- Use the switch  to turn the alarm On/Off.
- Tap the alarm time  to edit the alarm.




To adjust the alarm

Tap to select the hours or minutes and tap the red arrows to adjust the time. Tap Done  to go to the Alarm panel.



In the Alarm panel


Your new alarm is turned On automatically.

Tap Done  to save the alarm and go to the watch face.



On your watch face



The alarm hand is shown on the watch face (here in yellow).

Use the switch  in the Alarm panel to turn it On/Off.

Snoozing or stopping



When the alarm goes off


- Tap  to snooze the alarm for 9 minutes.
- Tap  to stop the alarm and return to the watch face.





When you are snoozing

A window appears briefly on your watch face to confirm your choice.

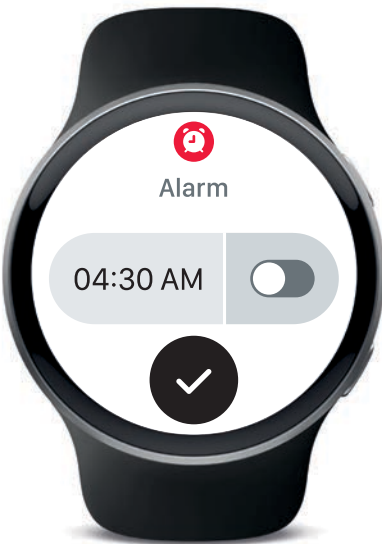Turning the alarm On/Off



In the Alarm panel

- Tap the switch  to turn off the alarm.
- Tap Done  to save your setting and go to the watch face.



On your watch face

The alarm indicator has disappeared from your watch face.

Using Night Mode

Night Mode reduces eye strain and glare, making it easier to focus on the text in low-light environments.

To activate Night Mode, press the **Back key** and tap **Night Mode** in the Menu.



To activate Night Mode

Tap to select **Night Mode** in the Menu to activate it.

Your watch will feature a dark and dimmed watch face.

To deactivate Night Mode

Press the **Back key** to deactivate Night Mode. The watch will return to your original screen settings and open the menu.

Managing notifications

When a Visit Transmitter or the mobile app is activated, it signals your watch that alerts you with vibrations and displays a message. Your transmitter setup decides what type of notifications you will receive.



Notification example

When e.g. the mobile phone rings, your bridge is activated and signals your watch.

On your watch face

Your watch alerts you with gentle vibrations and displays a message. Press the **Home key**, **Back key** or tap the screen to silence it.

Notification history

To get an overview of your recent notifications, tap the notifications icon on the watch face. Your notifications are sorted by date and time, and you can go back 6 notifications to see what has happened.



On your watch face

The **3** on the dial shows the number of missed notifications.

Tap the notifications icon to view the history list.

Viewing the history

Tap the top and bottom arrows to view your recent notifications.

The notifications are listed in chronological order.

Turning off a notification

Do you sometimes feel overwhelmed by too many notifications? Tap a notification to turn it off temporarily.

First, press the Back key to open the menu. Then select **Settings** and tap **Notifications** to open the panel.



In the Notifications panel

Tap the top and bottom arrows to find the notification you want to turn off.

To turn off a notification

Tap the notification you want to turn off. It will be greyed out. Tap it again to turn it back on.

Adding a transmitter

When you add a transmitter to your system, the associated notification will automatically be added to the Notifications panel.

Learn more

Read the transmitter’s user manual to learn how to add it to your system.



To add a device:

Push the **Test buttons** on the transmitter you want to add.
In this example we are adding a Visit Baby Monitor.

On your watch face

Your watch starts to vibrate, and the associated notification appears on your watch face.

To acknowledge

Press the **Home key**, **Back key** or tap the screen to silence it.
Your watch will return to the watch face.

In the Notifications panel

The first time you activate a transmitter, the associated notification will be added automatically.

Removing a notification

When you remove a transmitter from your system, it’s a good idea to remove the associated notification from the Notifications panel.

So for instance, if you remove a Baby Monitor, you might want to remove the Baby notification in the list too.



To remove a notification

Press the **Back key** to open the Menu and tap Settings .

In the Settings menu

Tap **Notifications** to open the Notifications panel.

In the Notifications panel

Tap the top and bottom arrows to find the notification you want to remove.

To remove the notification

Hold down the notification to remove it. A bin will appear. Tap the bin to confirm.

Settings menu

In the Settings panel you can adjust a multitude of parameters to make your Watch Receiver match your specific needs.

To open the Settings screen, press the **Back key** to open the Menu and tap **Settings** ⚙️.

Overview

Screen 1

- ⌚

Time and date
Set the time, date and year manually.
- 🔔

Notifications
Browse the list and turn off notifications temporarily.
- 🕒

Watch faces
Select the watch face that matches your style.
- ☀️

Screen brightness
Adjust the light intensity in 5 steps.
- ➡️

Next
Tap the symbol to access more parameters.



- Home key

Press the Home key to go back to the watch face.
- Back key

Press the Back key to go back to the Menu.

Overview

Screen 2

- 🕒

Alert timeout
Controls the duration of your vibrating alerts.
- 🔒

Safe Mode
Restricts access to advanced features.
- ⌚

Screen wake-up
Controls how you wake your watch display.
- ℹ️

About the watch
Shows the device ID, version information etc.
- ⬅️

Back
Tap the symbol to go back to the first screen.



- Home key

Press the Home key to go back to the watch face.
- Back key

Press the Back key to go back to the previous Settings screen.

Settings

This section describes the various Settings parameters and how to adjust them to fit your preferences.

Please note: Changing the Alarm timeout will not have any effect on safety critical notifications.

To open the Settings screen, press the **Back key** to open the menu and tap **Settings** ⚙️.

Setting the date and time




On the Settings screen
Tap the symbol to select the parameter you want to edit or tap Next ➡️ to go to the following Settings screen.




To set the date and time
Tap to select **Date and time** ⌚. Select the hours or minutes and tap the red arrows to adjust the time. Tap Next ➡️ to set the year.



Set the year
Tap the red arrows to set the year and tap Next ➡️ to set the date.

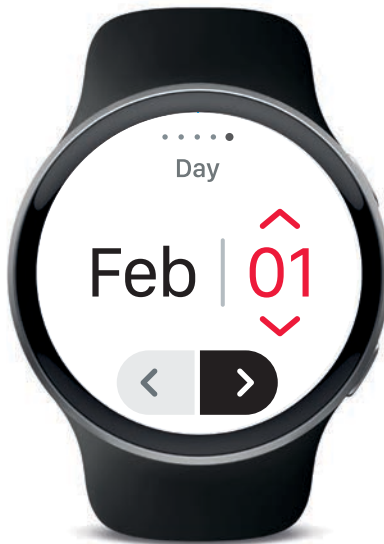

Set the date
Select the month or day and tap the red arrows to set the date. Tap Next ➡️ to finish.

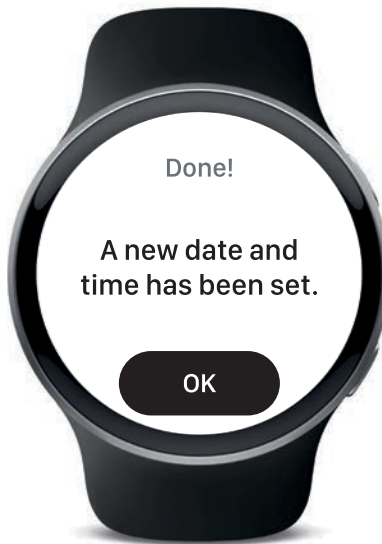

Done!
A screen appears to confirm that a new date and time has been set. Tap **OK** to return to the Settings screen.

Brightness



To adjust the brightness
Select **Brightness** ⚙️ on the Settings screen. Use the buttons to adjust the brightness and tap Done ✓.

Alert timeout



To adjust the alert timeout
Select **Alert Timeout** ⌚ on the Settings Screen. Choose between 10, 20 and 30 seconds and tap Done ✓.

Screen wake-up




To turn on Screen wake-up
Select **Screen Wake-Up** ⌚ on the Settings screen. Use the switch ➡️ to turn it On/Off and tap Done ✓.


Settings

Do you prefer a watch that only alerts you to important signals, and don't care much about other features? Turn on Safe Mode and your watch will be even easier to handle. To open the Settings screen, press the **Back key** to open the Menu and tap **Settings** ⚙️.


Using Safe Mode

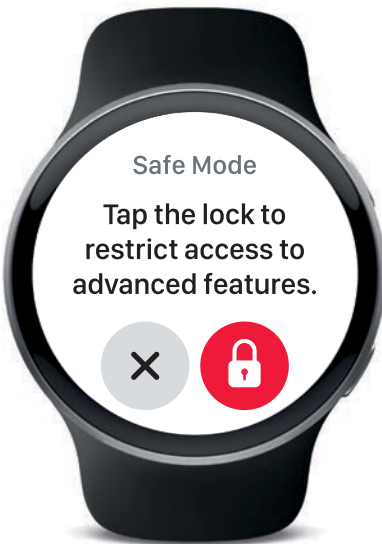

To activate Safe Mode
Select **Safe Mode** 🔒 on the Settings screen and tap the lock 🔒 to activate it.

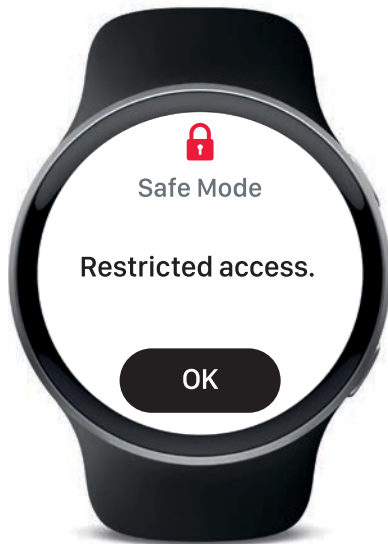

Safe Mode dialog
When Safe Mode is on, a dialog appears if you try to access advanced features.

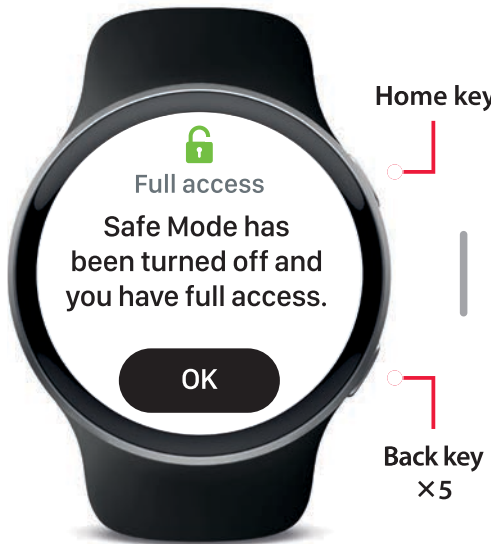

To deactivate Safe Mode
Hold down the Home key while pressing the Back key five times. Tap **OK** to return to the watch face.

Factory reset



To return to factory settings
Hold down both keys for 7s to initiate a factory reset. Tap **YES** to confirm. The current pairing data will also be removed.

Signaling for attention

To signal for attention, hold down the **Back key** for about 4 seconds. The Visit receivers around your home will sound, flash or vibrate to notify your family members.


Please note: The first time you signal for attention, the app will only add it to its Notifications list. Send the notification once more to ensure the signal is received on all devices.



To signal for attention:
Hold down the **Back key** for 4 seconds to signal for attention.



During the countdown
Tap **Stop** or press any key before the countdown has reached 1 to cancel and return to the watch face.

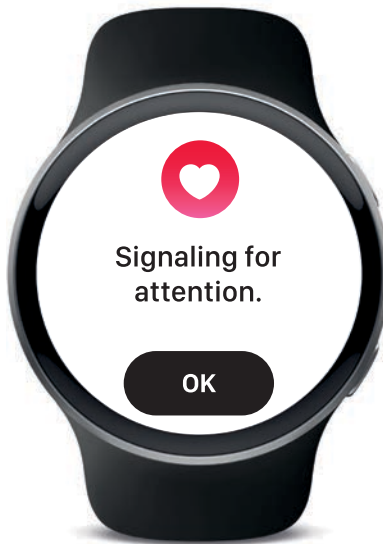

Signaling for attention
Tap **OK** to acknowledge and return to the watch face. If you don't tap OK, it will return to the watch face after 30s.

About the watch



To open the About panel
Select **About** ⓘ on the settings screen to open the About panel. Tap **Close** to return to the Settings screen.

Customizing the watch face

You can customize the watch face to suit your various styles. Do you want a numeric or a minimalistic look? The watch offers a selection of designs to choose from.

To open the Settings screen, press the **Back key** to open the Menu and tap **Settings** ⚙️.



To change the watch face

Select **Watch Face** ⌚ on the Settings screen. The current watch face is fronted in the gallery.

Find the watch face you like

Use the left < and right > arrows to toggle between the available watch faces.

Select dark or light version


Use the switch ☀️🌙 to toggle between the dark and light version of the watch face. Tap the watch face to select it.

Showing the new face

The selected watch face is now featured on your home screen.


Changing the band

Your watch can be fitted with virtually any band that features a lug width of 20 mm. Use the method below to attach a new band.

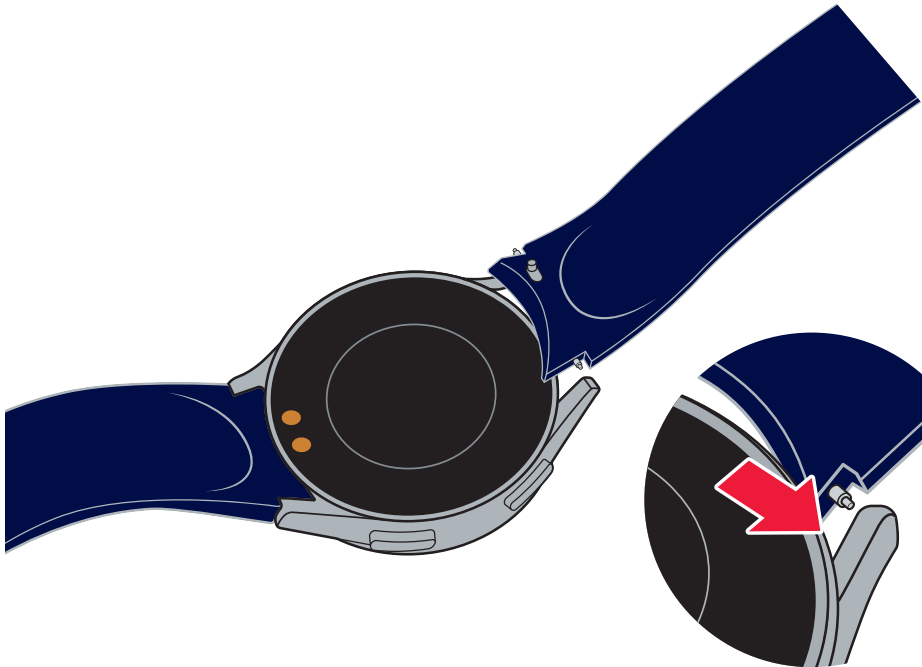

Connect the spring bar

Insert one end of the band's spring bar into the watch's lug.

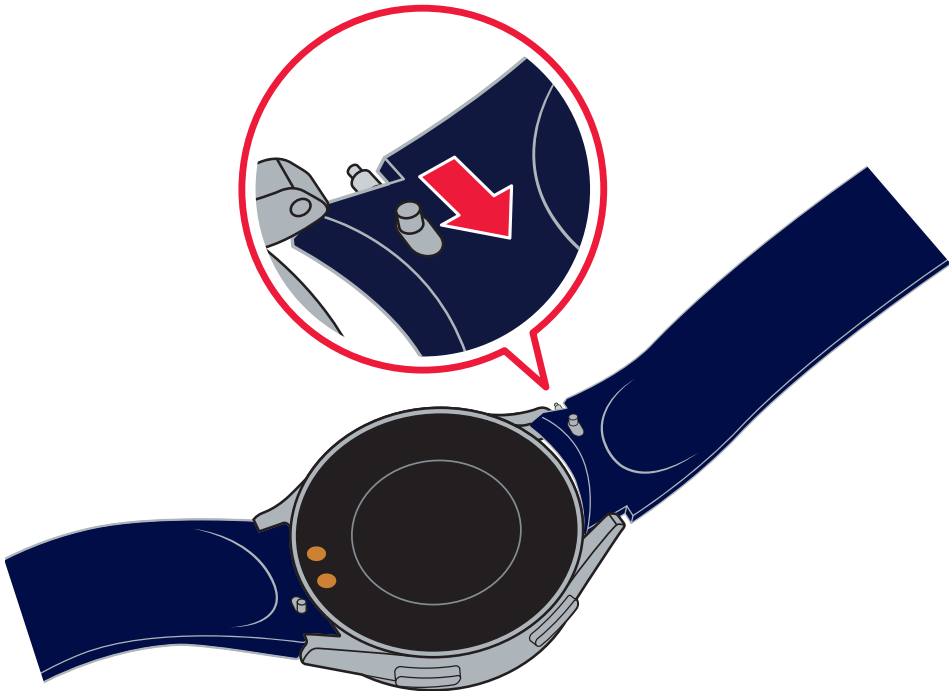

Connect the other end

Slide the spring bar inwards and connect the band.

Available watch faces

Minimalistic
Light



Classic
Light



Numeric
Light



Dressy
Light



Chronograph
Light



Diver
Light



Minimalistic
Dark



Classic
Dark



Numeric
Dark



Dressy
Dark



Chronograph
Dark






Diver
Dark



Troubleshooting

If you encounter any problem with your Watch Receiver, please try the following suggestions.

If	Try this
An Out-of-range symbol  appears on the watch face.	The symbol warns that your watch is temporarily out-of-range and that you will not receive any notifications. Move to an area that is closer to your notification system.
A Do Not Disturb symbol  appears on the watch face.	This symbol means that your notifications are temporarily silenced. Tap the symbol to turn the notifications back on. Please note that critical notifications like e.g. smoke and CO cannot be turned off.
The watch does not turn on	The battery on your watch is depleted. Attach your watch to the charging dock to charge the battery, see 'Charging your watch'. It can take up to 2 hours to fully charge the watch.
The touchscreen responds slowly or improperly	<ul style="list-style-type: none">▪ Make sure you are not wearing gloves, and that your hands are clean.▪ Make sure you are not using any sharp objects or only touching the screen with your fingernails.▪ Please note that humid conditions or exposure to water may prevent the touchscreen from function properly. If you have very dry skin, try to use some skin lotion to improve the touch response.▪ Ensure that your watch software is updated to the latest version.▪ If the screen is scratched or damaged, contact an authorized service center for assistance.
The watch is frozen and unresponsive	<ul style="list-style-type: none">▪ Disconnect your watch from the charger and let the battery run out until it's completely dead.▪ Reconnect your watch to the charger and wait for at least 30 minutes.▪ Press and hold the Home key to start your watch.▪ If the problem persists, contact an authorized service center for assistance.
I was not able to add a device	<ul style="list-style-type: none">▪ Bring your watch close to the bridge and make sure the watch is turned on.▪ Press the top Test button on the bridge to make sure the connection is not lost. Bring the device you want to add nearby, make sure it's turned on, and try to connect it again, see 'Adding a device'.

If	Try this
The watch is vibrating for no apparent reason	There is probably another Bellman alerting system installed nearby that triggers your system. Change the radio key on all units, see 'Changing the radio key' in the Bridge Transceiver user manual.
The watch does not vibrate or light up a symbol when a transmitter is activated.	<ul style="list-style-type: none">▪ If an Out-of-range symbol  blinks on the watch face, move to an area that is closer to your Bellman system. Obstacles, such as walls or electrical equipment close to the devices can reduce the Bluetooth coverage area significantly. Examples of electrical equipment includes, but is not limited to, microwave ovens, Wi-Fi routers and plasma TV screens. If the problem remains, make sure that the bridge is connected to power, and that the backup batteries are fresh. To test the connection, press the top Test button on the bridge. A message will appear on your watch to confirm that the connection is OK.▪ If no Out-of-range symbol appears on your watch face, there are two possibilities:<ol style="list-style-type: none">1 The connection between the transmitter and the bridge is lost. Move the units closer together. To test the connection, press the Test button(s) on the transmitter. If the bottom LED on the Bridge lights up in green, the connection between the transmitter and the bridge is OK.2 The transmitter doesn't trigger correctly. Check all connections to the transmitter, and make sure that the batteries are not depleted.
The watch is hot to the touch	<ul style="list-style-type: none">▪ This is normal and should not affect your watch's lifespan or performance. If your watch overheats or feels hot for a prolonged period, do not use it for a while.▪ If your watch continues to overheat, contact an authorized service center for assistance.
The watch battery doesn't charge properly	<ul style="list-style-type: none">▪ Make sure that the charging dock is connected to power and that you have attached your watch to the charging dock correctly, see 'Charging your watch'.▪ A charging symbol appears, and short vibration occurs to confirm that charging has started. If the display is turned off, press any key to check the charging status.

Safety and handling

This section contains information about safety, handling and operating conditions. Keep this booklet for future use. If you are a professional installer, this booklet should be given to the householder.

Hazard warnings

- Failure to follow these safety instructions could result in fire, electric shock, or other injury or damage to the device or other property.
- Your device is not a toy. Do not allow children to play with it because they could hurt themselves and others, or damage the device. Keep the device and all its accessories out of the reach of small children.
- Be aware that alarms and notifications can be missed if batteries run out.
- This device should not be used as the only means of security in life threatening situations.
- This product is equipped with a USB cable for connecting to a desktop or notebook computer or to the charger. Be sure your computer is properly grounded before connecting your product to the computer, and that the outlet is installed and grounded in accordance with all local codes and ordinances.
- Do not use or store this device near any heat sources such as naked flames, radiators, ovens or other devices that produce heat.
- Do not dismantle the device; there is a risk of electric shock. Tampering with or dismantling the device will void warranty.
- This product is water and dust-resistant according to the IP67 Rating, which tests submersion in water at up to one meter for up to 30 minutes. It is not shockproof.
- Protect the device from shocks during storage and transport.

- Do not make any changes or modifications to this device. Use only original Bellman accessories to avoid any electrical shock.

Information on product safety

- Failure to follow these instructions could result in damage to the device and void the warranty.
- Prevent the charger port and the small end of the charger cable from coming into contact with conductive materials, such as liquids, dust, metal powders, and pencil leads.
- While charging, the device and the charger may become hot. This does not affect the device’s lifespan or performance and is in the device’s normal range of operation.
- Don’t use excessive force or a metallic object on the touchscreen. It may damage the tempered glass surface and void the warranty.
- Do not allow the touchscreen to come into contact with other electronic devices. Electrostatic discharges can cause the touch screen to malfunction.
- Do not drop your device. Dropping onto a hard surface can damage it.
- Do not insert foreign objects into the product.
- Do not place the product in a microwave oven.
- Do not use the device in areas where electronic equipment is prohibited.
- The device may only be repaired by an authorized service center.
- No company, enterprise or user is allowed to change the frequency, enhance the transmitting power or alter the original characteristics or performance on an approved low power radio frequency device.
- This device should not be used on aircrafts unless specifically permitted by flight personnel.

Using it with a pacemaker

Persons with pacemakers should:

- ALWAYS keep the product more than six (6) inches from their pacemaker when the product is turned ON.
- Not carry the product in a breast pocket.
- Use the wrist opposite the pacemaker to minimize the potential for RF interference.
- Turn the product OFF immediately if there is any indication that interference is taking place.
- If you use any other personal medical device, consult your physician or the manufacturer of your device to determine if it is adequately shielded from external RF energy.

Battery warnings

- Do not disassemble, open, crush, bend, deform, puncture or shred.
- Do not modify or re-manufacture, attempt to insert foreign objects into the battery, immerse or expose to water or other liquids, expose to fire, explosion or other hazards.
- Make sure that no sharp-edged items, such as animal’s teeth or nails, come into contact with the battery. This could cause a fire.
- Only use the battery for the system for which it is specified.
- Only use the battery with a charging system that has been qualified

with the system per CTIA Certification Requirements for Battery System Compliance to Safety information 7IEEE1725. Use of an unqualified battery or charger may present a risk of fire, explosion, leakage, or other hazards.







- Do not short circuit a battery or allow metallic conductive objects to contact battery terminals.
- Do not attempt to replace the battery.
- Battery usage by children should be supervised.
- Avoid dropping the watch. If the watch is dropped, especially on a hard surface, and you suspect damage, take it to a service center for inspection.
- Improper battery use may result in a fire, explosion, or other hazards.
- Always unplug the charger from the wall socket after the product is fully charged to save unnecessary power consumption of the charger.
- There is risk of explosion if the battery is replaced with an incorrect type.

Warning Notice for battery replacement

- For your safety, do not remove the battery incorporated in the product. If you need to replace the battery, take it to the nearest authorized service center for assistance.
- A Li-Po battery is a hazardous component which can cause injury.
- Battery replacement by non-qualified professional can cause damage to your device.

Notes on water resistance

This product is water and dust resistant in compliance with the Ingress Protection rating IP67. Splash, water, and dust resistance are not permanent conditions and resistance will decrease over time as a result of normal wear. Liquid damage is not covered under warranty, but you might have rights under consumer law. Do not use the product in the following environments:

-  Do not immerse the product in any liquid chemicals, like soap, etc. other than water.
-  Do not immerse the product in salt water, such as sea water.
-  Do not immerse it in hot spring or wear it when you take hot showers.
-  Do not swim wearing this product or use it under water.
-  Do not expose the product to extensive heat like e.g. in a hot car.
-  Do not place the product directly on sand (such as at a beach) or mud.

- Once the product gets wet, water may remain in gaps of the main unit. Shake the product well to drain water. Even after water is drained, the inner parts might be still wet. Although you can use the product, do not place it near things that must not get wet.
- Do not use the product in places where it may be sprayed with high-pressure water (e.g. near a faucet or shower head) or submerge it in water for extended periods of time, as the product is not designed to withstand high water pressure.
- This product is not resistant to shock. Dropping the product might damage or deform the unit causing water leak.

General guidelines

While using your product, be aware of the following common-sense guidelines:

- Use and store your product in temperatures between 0° to 35° C, 32° to 95° F. Exposing your product to extremely low or high temperatures may result in damage, malfunction or even explosion.
- Using a mobile device or accessory while driving may cause distraction. Please keep your eyes on the road.
- Always follow any special regulations in a given area. Turn your device off in areas where use is forbidden or when it may cause interference or danger.
- Do not to charge the watch until it reaches low levels. Charging to frequently will lower the overall battery lifetime.

Advice on how to avoid skin irritation

When you wear the watch for a long time, skin irritation may occur due to certain factors, such as friction, pressure, or moisture. If you have been wearing your watch for a long time, remove it from your wrist for a while to keep your skin healthy and allow your wrist to rest.

Skin irritation may occur due to an allergy, environment factors, other factors, or when your skin is exposed to soap or sweat for long periods. In this case, stop using your watch immediately and wait 2 or 3 days for the symptoms to ease. If the symptoms persist or worsen, call your doctor.

Make sure your skin is dry before wearing your watch. If you wear a wet watch for a long time, your skin may be affected.

Maintenance and warranty

This section contains information about maintenance, service and warranty conditions.

Cleaning

Wipe your Watch Receiver clean with a nonabrasive, lint-free cloth. If necessary, lightly dampen the cloth with fresh water. If your Bellman Watch has substances or debris visible in places such as the ports, you can hold your Bellman Watch under lightly running, warm, fresh water until it's clean. This device does not require sterilization.

Service and support

If the device appears to be damaged or doesn't function properly, follow the instructions in the user guide and this leaflet. If the product still doesn't function as intended, contact an authorized service center for information on service and warranty.

Warranty conditions

Bellman & Symfon guarantees this product for two (2) years from date of purchase against any defects that are due to faulty materials or workmanship. This guarantee only applies to normal conditions of use and service, and does not include damage resulting from accident, neglect, misuse, unauthorized dismantling, or contamination howsoever caused. This guarantee excludes incidental and consequential damage. Furthermore, the warranty does not cover Acts of God, such as fire, flood, hurricanes and tornadoes.

This warranty gives you specific legal rights and you may also have other rights that vary with territory. Some countries or jurisdictions do not allow the limitation or exclusion of incidental or consequential damages, or limitations on how long an implied warranty lasts, so the above limitation may not apply to you. This guarantee is in addition to your statutory rights as a consumer. The above warranty may not be altered except in writing signed by both parties hereto.

Technical specifications

- Dimensions: 49.5 x 44.5 x 9.3 mm, Weight: 42g
- Screen Size: 1.3" TFT full touch screen
- Screen Resolution: 240x240px
- Battery Power: 180mAh
- Operating time: Up to 1 week depending on use
- Material: ABS+TPU
- Waterproof: IP67
- Charger: USB type A charger
- Bluetooth frequency: 2400 – 2483.5 MHz
- Coverage: Up to 250 m open range. Obstacles, such as walls, bodies or electrical equipment close to, or between the devices can reduce the Bluetooth coverage significantly.

In the box

- BE3330 Watch Receiver
- BE9419 Wrist band, black
- BE9292 Pocket charger
- Getting started guide

Correct disposal of this product

This product and its electronic accessories (e.g., charger and USB cable) should not be disposed of with other household waste.

To prevent possible harm to the environment or human health from uncontrolled waste disposal, please separate these items from other types of waste and recycle them responsibly to promote the sustainable reuse of material resources.

We recommend household users to contact either the retailer where they purchased this product, or their local government office, for details of where and how they can take these items for environmentally safe recycling.

We recommend business users to contact their supplier and check the terms and conditions of the purchase contract. This product and its electronic accessories should not be mixed with other commercial wastes for disposal.

Correct disposal of batteries in this product

The batteries in this product should not be disposed of with other household waste at the end of their working life.



Manufacturer

Head Office

Bellman & Symfon Group AB
Södra Långebergsgatan 30
436 32 Askim
Sweden
Phone: +46 31 68 28 20
E-mail: info@bellman.com

Distributor | Distribuidor | Distributeur

USA & Canada

Bellman & Symfon North America Inc.
PO Box 1388, Morrisville
North Carolina 27560, USA
Phone: (910) 239-5796
Customer support: (833) BELLMAN
or (833) 235-5626

Australia & New Zealand

Word of Mouth Technology
PO Box 1001
Croydon VIC 3136
Australia
Phone: 03 9723 0660
Email: info@wom.com.au